ergo
STRAIGHT
by platinum

platinum
STAIRLIFTS

USER GUIDE 

MODE D'EMPLOI 

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GUIDA UTENTE 

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Please keep this guide in a safe
place for future reference.

USER GUIDE

Please keep this guide in a safe place for future reference.

Original Document

P20

P36

P52

P68

CONTENTS

6	Get to Know Your Lift	11	Problems & Troubleshooting
7	Controls	12	Status Display Codes
8	Charging Points & Remote Controls	14	Assisting a Stranded User & Hand Winding The Stairlift
9	How to Use Your Stairlift	15	Technical Information
10	Cleaning & Maintenance	17	Installation & Service History

THANK YOU FOR CHOOSING PLATINUM STAIRLIFTS

Welcome to your Platinum Ergo Straight User Guide. Our range of Platinum stairlifts have been designed to offer you the freedom and comfort to live independently in the home you love.

Please take the time to read this manual carefully and familiarise yourself with the lift functions. We recommend that you share this information with anyone else who might need to use the lift, or any other regular visitors, such as cleaners or carers.


Key things to help you look after your stairlift:

- Ensure your stairlift is kept plugged in and switched on at all times. Never allow the mains socket allocated to this equipment to be used for any other purpose.
- Always keep the area underneath and around your stairlift free from obstructions, as this could result in the stairlift malfunctioning.
- This stairlift has been designed to carry one person only and has a maximum weight limit of 25st (160kg) on a standard rail.
- Your stairlift has been designed to run continuously for no more than 15 minutes in any three hour period. If you use the stairlift too often without taking a break, the batteries will not have sufficient time to recharge.

Your local Stairlift Service Centre:
For technical issues or servicing please call:
Please keep this guide in a safe place for future reference.

GET TO KNOW YOUR LIFT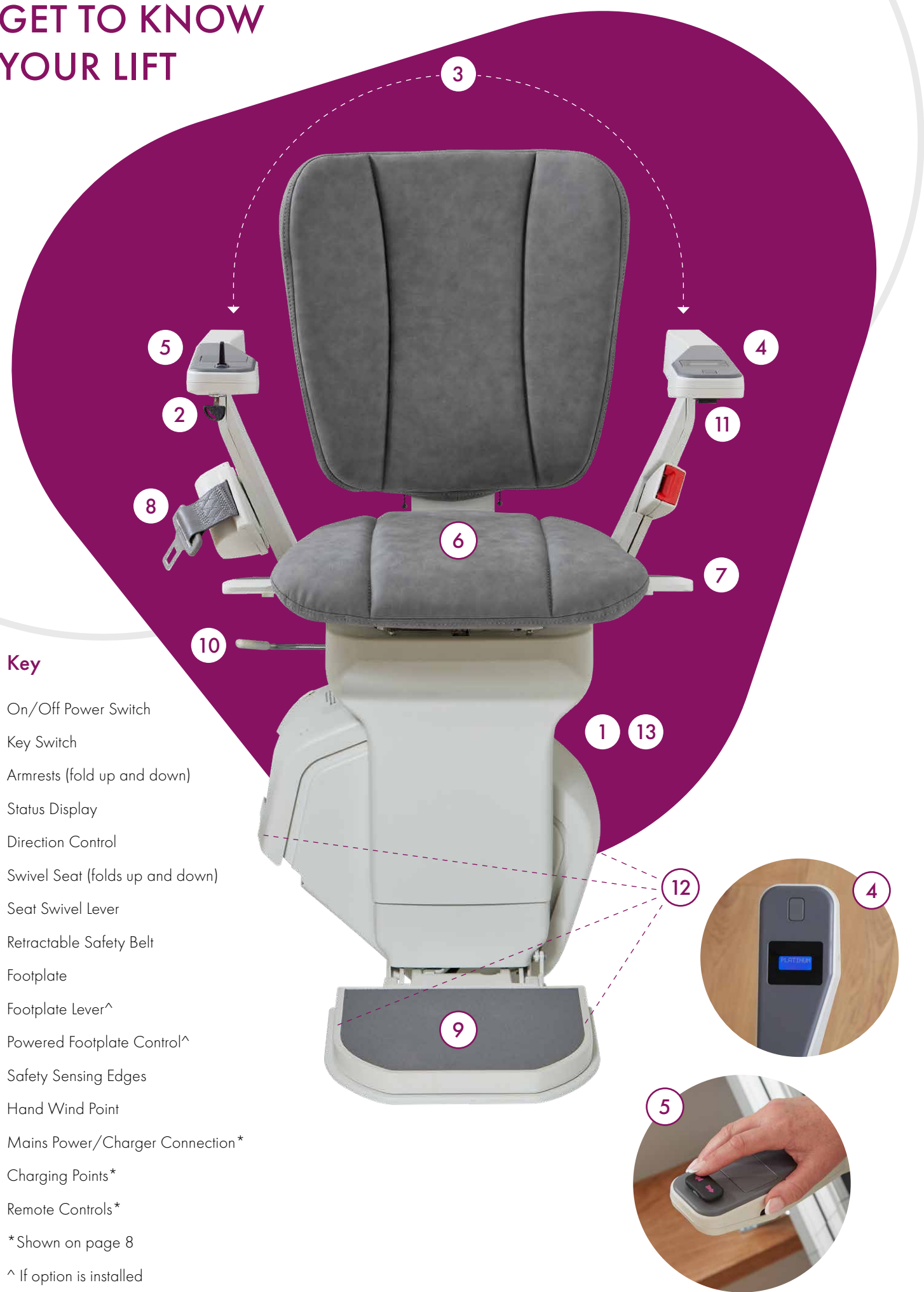

Key

- 1. On/Off Power Switch
- 2. Key Switch
- 3. Armrests (fold up and down)
- 4. Status Display
- 5. Direction Control
- 6. Swivel Seat (folds up and down)
- 7. Seat Swivel Lever
- 8. Retractable Safety Belt
- 9. Footplate
- 10. Footplate Lever^
- 11. Powered Footplate Control^
- 12. Safety Sensing Edges
- 13. Hand Wind Point
- 14. Mains Power/Charger Connection*
- 15. Charging Points*
- 16. Remote Controls*

*Shown on page 8

^ If option is installed

CONTROLS

1 On/Off Power Switch

If the stairlift is turned off here, it will not recharge.

2 Key Switch

To stop others from operating the stairlift, use the key switch located underneath the armrest, turn to remove the key. This will not affect the charging capabilities.

3 Armrests

These can be folded away at the end of the journey to keep the staircase clear.

4 Status Display

Displays the current status. If any faults develop with the operation of the lift they will be displayed here. See status display codes section - page 12.

5 Direction Control

When seated, move the lift in your desired direction by pushing and holding the direction control at the tip of the armrest. There will be a short delay before the stairlift will start or change direction. Release the control to stop the chair from moving.

6 Swivel Seat including lever 7

The seat can rotate at the top of the staircase to aid a comfortable entrance and exit from the lift. In the case of the Manual Swivel, the seat can also be partially rotated at the bottom of the staircase. For your safety, the seat must be locked into its normal travel position before you can travel. Always make sure it is locked before you sit down or stand up from it.

Manual Swivel Option

To rotate the seat, lift one of the levers (7) at the side of the seat. Allow the seat to rotate until it stops, then release the lever to lock the seat into place, this must be done when the stairlift is stationary. If you attempt to do this while traveling, the lift will stop.

Powered Swivel Option

If your stairlift is fitted with our power swivel option, when reaching the top of the stairs, keep the control device pressed. After a short pause, the chair will swivel towards the landing. To return the chair to the normal travel position, press down on your control device and to swivel the chair before making the journey downwards.

8 Retractable Safety Belt

Pull the safety belt from the holder, across your lap and clip it into the socket. Make sure you use this for every journey. Do not unclip until the stairlift has stopped, this includes when the chair is swivelling into position. The lift will not travel without the safety belt fastened.

9 Footplate including 10 & 11

The footplate can be folded up and down using the footplate lever (10).

Powered footplate option


If you have chosen to have a powered footplate fitted, the button to operate this is under the left hand arm (11).

12 Safety Sensing Edges

If the lift encounters an obstruction, it will stop. Release the control device and wait for approximately three seconds before reversing away from the obstruction.

CHARGING POINTS & REMOTE CONTROLS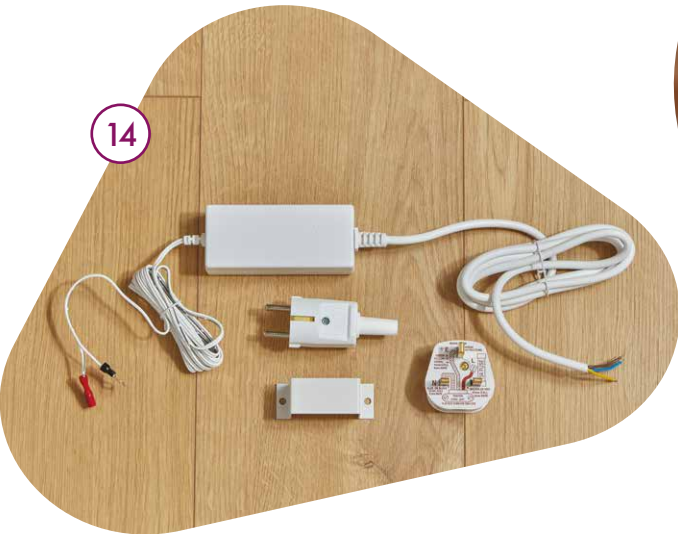

14 Mains Power/Charger Connection

The stairlift should be plugged in at the nearest mains socket. This could be at the top or bottom of the stairs. The lift needs to be plugged in and switched on at all times, the chair will charge when stationary on a charging point

15 Charging Points

The charging points are located at the top and bottom of the rail there may also be an intermediate parking point on your rail. The lift should be parked on a charging point when it is not in use. If the charger to the lift is switched off or the chair is not parked correctly at a charging point, there will be an audible beep, and 'OFF CHARGE' will show on the status display.

16 Remote Controls

There are three control options on the remote:

- 1. UP - Red upwards arrow
- 2. DOWN - Red downwards arrow
- 3. PARK - Grey home icon

To call your stairlift towards you, press the controller in the direction you need the lift to travel using the arrow buttons. Hold the button until the lift stops and swivels into position.

To send your stairlift to the nearest charging position, press the grey home button. Hold the button until the lift reaches its charging point and comes to a stop.

Before you use your stairlift, please read all of these instructions carefully.

- This stairlift has been designed to carry one person in a seated position.
- Do not allow children to play on or near the stairlift.
- The boarding/exit area of the stairlift must be kept well lit (to at least 50 Lux).
- If you are not able to safely get on or off this lift independently, seek assistance from someone who can help you.
- When you are not using the stairlift, please park it on one of the charging points (15).

HOW TO USE YOUR STAIRLIFT

Getting On The Stairlift

- 1. Fold down the footplate (9).
- 2. Fold down the seat if it is raised, and ensure it is locked in the correct position so you can sit on it safely. (6)
- 3. Fold down the armrests (3).
- 4. Sit on the seat and fasten the safety belt (8).
- 5. If required, swivel the seat back into the correct traveling position and place your feet firmly on the footplate (9).

Travelling Up Or Down The Stairs

- 1. Keep your feet on the footplate (9) and your arms on the armrests (3).
- 2. Press and hold the direction control (5) in the direction you wish to travel. There will be a short delay and then the stairlift will slowly move off. The stairlift may adjust its speed as it travels along the rail, to ensure a smooth and comfortable journey.
- 3. Keep the direction control pressed in the direction of travel until the stairlift comes to a complete stop. For lifts with a power swivel, ensure that the seat has swivelled into the correct position to exit the lift.
- 4. To stop the lift during travel or change direction, release the direction control, wait for a few seconds and then operate the control in the direction you wish to travel. After a short delay, the lift will move off.

Getting Off The Stairlift

- 1. The seat must be in a position for you to get off it safely. This may mean swivelling the seat. On a manual swivel this is done by moving either of the seat swivel levers up. (7)

If your lift is fitted with the powered swivel option, keep your finger pressed on the control device and the lift will automatically swivel away from the steps.
- 2. Unfasten the safety belt and get out of the seat. (8)
- 3. If it is required, fold up the seat and armrests (3) and footplate (9).

Remote Control

- 1. To move the stairlift without a passenger, or to assist a seated passenger, press and hold down the appropriate direction button on the remote control (16).
- 2. Use the PARK button on the remote control to send the lift to a charging point. The lift can only be sent upwards to the nearest point, if you want it to come downwards you must press the DOWN arrow.
- 3. If the stairlift is to be moved without a passenger, the seat must be returned to its correct traveling position. The safety belt is not required when operating the lift from a remote control.

PROBLEMS & TROUBLESHOOTING

If you experience a problem, here are some useful guidelines to follow before calling out an engineer. Please check:

1. The key switch (2) is turned to the ON position.
2. The lift charger is plugged into the wall socket and the socket is turned on. If the lift is not charging, the message 'OFF CHARGE' will be shown on the status display (4).
3. There is no physical obstruction stopping the lift from moving. If there is, a message will be shown on the status display to reflect this. See status display codes - page 12.
4. The seat is in the proper travelling position – on a manual swivel model the chair will not move if the seat is swivelled.
5. A different control device (16), if it works there is a possibility that a battery needs replacing in one of the controls.

To change batteries in remote controls


1. Open the rear cover of the remote control and remove the old batteries.
2. Replace them with two new AA batteries, ensuring polarity is correct.
3. Replace cover.

CLEANING & MAINTENANCE

Cleaning

- Before cleaning your stairlift, ensure it is switched off using the key switch (2).
- The stairlift must be kept clean to ensure continued reliable operation. The plastic, metal, and painted metal areas of your lift can be cleaned using a small quantity of mild, non-abrasive surface cleaner. Apply your cleaner to a soft cloth and use a different one to wipe away any residue.
- The upholstery is resistant to most household stains, many of which can be wiped off easily. Regular cleaning with a damp soapy cloth or sponge, followed by a wipe with a damp clean cloth will help the upholstery maintain a fresh appearance.
- **Important Information** - Solvent based cleaners, bleaches, abrasives, synthetic detergents, wax or silicone based polishes and aerosol sprays should not be used on any part of the stairlift. Do not attempt to clean the interior of the stairlift or the greased area on the rail. If specialist cleaning is required, please contact your local service centre.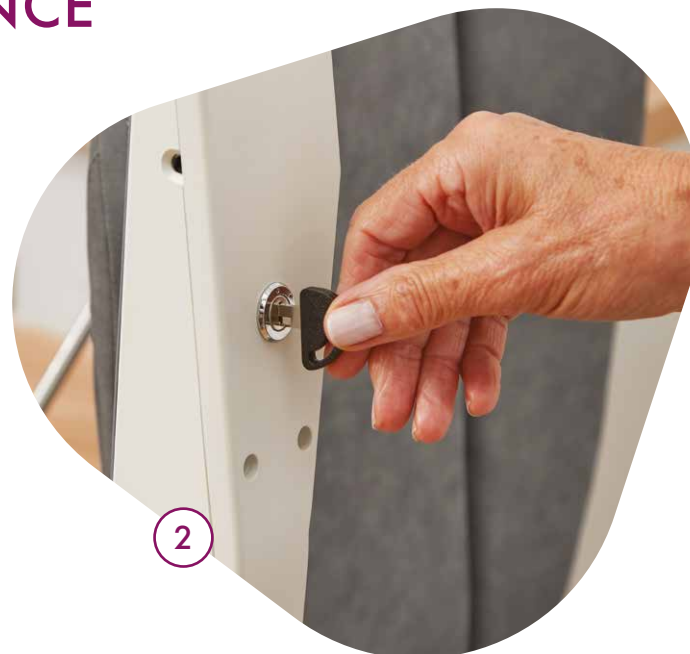

Maintenance

- To maintain safe and reliable operation, we advise your stairlift is serviced and safety inspected every 12 months. Your local stairlift service centre will be pleased to discuss servicing your stairlift and any warranty arrangements.
- Should you need them, replacement parts are available from your local stairlift service centre.

Mains Power, Battery Power and Power Cuts

Mains power charges a set of large internal batteries, These then power the motor, which moves you up and down the stairs.

In the event of mains power failure, you can continue to use the stairlift for a short while. The batteries will have stored enough energy to take you up and down the stairs a few times.

If the battery power runs low (for example, following a power cut), a ‘CRITICAL BATTERY’ message will be shown on the status display.

At this point, the stairlift batteries need to be recharged urgently. Do not use the stairlift if the ‘CRITICAL BATTERY’ message is reported in the status display other than to move DOWN to the nearest available charging point.

Battery Replacement

The large internal batteries will need to be checked and replaced periodically by your service engineer.

If you require an engineer visit and there is not a technical problem with the lift, the engineer may make a charge – this could apply even during the warranty period of the lift.

If you are still experiencing a problem, please make a note of the code on the status display, before contacting your local service centre and requesting a visit from an engineer, this will help your engineer to diagnose the problem quickly.

STATUS DISPLAY CODES

These messages are displayed on the status display (4).

MESSAGE	WHAT IT MEANS	ACTION TO TAKE
PLATINUM	OK. Stairlift on charge and should operate normally.	No action.
NO DISPLAY	No Power.	Check charger is plugged in and turned on at the wall socket. Check the on/off switch is in the “I” position.
OFF CHARGE	Lift is not charging. The lift has been stopped between charging points or charger is switched off at wall socket.	To charge, park on a charging point. Ensure the charger is switched on at the wall socket.
SEAT SWIVEL	Seat in swivelled position (power swivel only).	Lift can not move down the staircase until the powered swivel seat is returned to its normal travelling position.
RETURN TO TOP	(Powered Hinge Rails Only.) Shows when the lift is in between parking points for more than 1 minute. Also shows when leaving the intermediate parking point in the up direction after using the park button to stop on the intermediate parking point and raising the hinge.	In either case, move the lift up to the next available parking point and the lift will behave normally.

MESSAGE	WHAT IT MEANS	ACTION TO TAKE
HINGE	(Powered Hinge Rails Only) Shows when hinge is in transit. Also shows when parked on an intermediate parking point and the hinge is in the raised position.	No action.
CRITICAL BATTERY	Batteries urgently need charging. The lift will travel in down direction.	Move the lift to the next charge point in the downwards direction and leave for 1 hour.
SWITCH LIFT ON	Stairlift is parked on a working charge point but the on/off switch is in the ‘O’ (off) position. The stairlift will not charge up in this state unless the on/off switch is switched to the ‘I’ (on) position.	Check the on/off switch is in the I position.
MOVING LEFT	Lift travelling left.	No action.
MOVING RIGHT	Lift travelling right.	No action.
FINAL LIMIT CHECK KEY CHECK SWIVEL	These messages cycle.	1. Check the key is in the arm rest and switched on. 2. Check the seat is swivelled into the correct position.
EDGE	Lift has been stopped by a safety sensing edge.	Ensure that both the track and the staircase are free from obstructions.
END STOP	If the lift is at the end of the rail the display will read ‘END STOP’ until the controller is released. If the stairlift is not at the end of the rail it will have been stopped by the ‘END STOP’ safety sensing edges.	Ensure that both the track and the staircase are free from obstructions.
MULTIPLE CALLS	More than one controller is being pressed at the same time.	Check there are no buttons pressing in on remote controls.
BRAKE	Motor brake has been activated.	Turn charger off at wall socket and turn lift off. Wait for around one minute before turning everything back on. If the problem persists contact your local stairlift service centre.
RELAY FAULT	Power relays did not close.	1. Ensure the lift is parked on a charge point and charger is turned on. 2. Turn charger off at wall socket and turn lift off. Wait for around one minute before turning everything back on. If the problem persists contact your local stairlift service centre.

ASSISTING A STRANDED USER

If a user becomes stranded on the stairlift for any reason, they should be assisted from the chair in the following way:

- 1. Lift a manual swivel lever and rotate the chair so that the user faces up the stairs. **Never exit without turning the seat towards the staircase and ensuring it locks into position.**
- 2. Release the safety belt for the user to leave the stairlift. **The user should always exit upwards.**

Do not run this stairlift without the plastic panels properly fitted. Failure to adhere to this will cause severe damage to the stairlift that will not be covered by your product warranty.

HAND WINDING THE STAIRLIFT (13)

In exceptional circumstances, where the lift is causing an obstruction, you may be advised to hand wind the stairlift using the supplied winding handle.

The stairlift should only be hand-wound by, or under the supervision of, a trained engineer, this could be done over the phone. In some circumstances this can cause damage to the stairlift which will not be covered under your guarantee or warranty.

- 1. Turn off the stairlift and remove the circular plastic cover and Insert the hand-wind tool provided as pictured and locate onto the mechanism.
- 2. On a left-handed lift, rotate anticlockwise to move the lift upwards and on a right-handed lift rotate clockwise to move the lift upwards.

OUTSIDE INSTALLATION

Important information for Platinum Ergo Straights installed outside.

- If your lift has been designed for outdoor use please ensure that the supplied waterproof cover is used to cover the seat when the unit is not in use.
- If your outdoor lift is exposed to excessive amounts of water spray, from a broken roof gutter for instance, discontinue its use until the source of the excessive water has been attended to and repaired.
- Before putting the lift back into normal use it must be inspected by your local stairlift service centre.
- Note that on lifts intended for outdoor use, exposed metal parts will corrode over time. This corrosion will be accelerated if the lift is installed in coastal areas, due to the saline nature of the environment.
- Platinum Stairlifts recommends that lifts intended for outdoor use are serviced every six months. Your local stairlift service centre will be happy to discuss this with you in further detail.

TECHNICAL INFORMATION

Weight Limits

This stairlift has been designed to carry one person only, and has a maximum weight limit of 25st (160kg) on a standard rail.

Operating Periods/Excessive Use

Your stairlift has been designed to run continuously for no longer than 15 minutes in any three hour period. If you use the stairlift too often without taking a break, the batteries will not have sufficient time to recharge.

Upholstery

Carelessness with matches, cigarettes and so on can cause a fire. The upholstery material on your stairlift has been tested for compliance with the Furniture and Furnishings (Fire Safety) Regulations 1988 and with EN 1021-2:2014.

Maintenance

To maintain safe and reliable operation, your stairlift needs a regular safety inspection and service which should take place once every 12 months.

Your local stairlift servicing centre will be pleased to discuss servicing your stairlift, along with any warranty arrangements.

Sound Pressure Levels

The emission sound pressure level at the operators position is expected not to exceed 70dB(A).

CAUTION

- Any modifications which have not been expressly approved by the manufacturer may void the warranty and may cause damage. Your stairlift should be inspected and maintained by a Platinum approved service engineer.
- In the event of a fire, DO NOT use the stairlift.

EXAMPLE OF A CERTIFICATE FOR TEST & EXAMINATION

Certificate For Test & Examination After Installation				
Location of Installation	Make Serial No. Rated Load	Model Rated Speed		
Electrical Tests 1. Electrical supply voltage V AC 2. AC supply frequency Hz 3. Phase Single 4. Stall current Amps 5. Tripping time Seconds 6. Mains fuse rating Amps 7. Check mains polarity correct Yes / No 8. Motor reversal delay Seconds	Battery Tests 1. Battery Voltage V DC 2. Check battery polarity correct Yes / No 3. Confirm battery enclosure ventilated Yes / No 4. Confirm batteries securely fitted Yes / No 5. Confirm batteries fully charged Yes / No 6. Confirm no leak when batteries fitted Yes / No 7. Confirm chassis is effectively grounded Yes / No			
Installation / Operation Tests 1. Feet are correctly screwed to risers 2. Rail feet are correctly made 3. End stops are correctly attached to rail 4. End plates are correctly attached to rail 5. Ensure seat mounting bolts are tightened to torque 20Nm 6. Confirm all on/off switches operate correctly 7. Confirm all safety limits operate correctly 8. Confirm final limits operate correctly 9. Confirm safety gear operates correctly 10. Confirm swivel interlock switch operates correctly 11. Confirm hinge interlock operates correctly (if fitted) 12. Confirm off charge indicator operated correctly 13. State stopping distance of carriage after application of the brake 14. Confirm hand-winding operates correctly 15. State distance of footrest above floor level 16. State running current	Upper Upper Manual	Yes / No Yes / No Yes / No	Lower Lower Powered	Yes / No Yes / No Yes / No Yes / No / NA Yes / No / NA Yes / No mm Yes / No mm Amps
Handover 1. Confirm user manual handed to customer 2. Confirm stairlift demonstrated to customer 3. Confirm premises left clean and tidy 4. Balance collected				
Yes / No £..... Cash / Cheque / Credit Card				
Comments / Observations				
I / we certify that on / / this stairlift was thoroughly examined and found to be free from obvious defects.				
Name (Engineer)		Signed Company		
Name (Customer)		Signed		

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Platinum Stairlifts Ltd, Unit 10 Crosshills Business Park,